

Supplies

1. Bowl
2. Swabs
3. Gauze dressing

Inner cannulas should be cleaned at least daily and more often as needed to remove and prevent mucus build up and keep tube surfaces clean.



1 Prepare a bowl with tap water (drinkable), or if needed, sterile or distilled water.



2 Rinse your used inner cannula in the bowl. This can also be done under tap water.



3 Clean the inner cannula from both ends with the swab. Use as many swabs as needed for cleaning.



4 Use a swab to clean the outside of the inner cannula.



5 Rinse inner cannula in a clean bowl of water or under lukewarm tap water.



6 Leave to air dry.

Place in clean and dry storage container until next use.

Discard swab after each use.

Attention

Never insert the swab into the tracheostoma itself to clean. The inner cannula must always be removed from the outer cannula prior to cleaning.



Watch video
instructions here

Head office:

Atos Medical AB
Hyllie Boulevard 17
SE-215 32 Malmö, Sweden
Tel: +46 (0) 415 198 00
Email: info@atosmedical.com

Manufacturer:

Atos Medical AB
Kraftgatan 8
SE-242 35 Hörby, Sweden

Manufacturer:

Tracoe Medical GmbH
Reichelsheimer Straße 1/3
55268 Nieder-Olm, Germany

© Coloplast A/S. All rights reserved. Atos and the Atos Medical logo are trademarks of Coloplast A/S.

The information contained in this material does not replace the Instructions for Use of the products demonstrated. You must see your physician, clinician, or qualified caregiver for advice on your condition and on products that may be appropriate for you.

MC3020-TcEN 202305