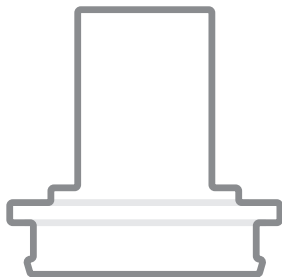
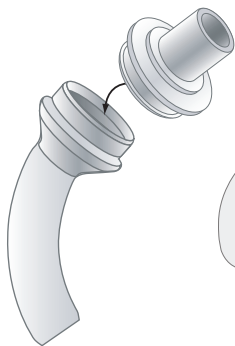
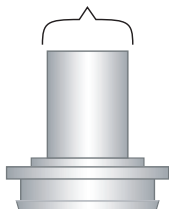
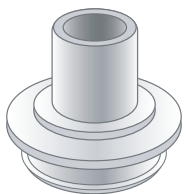
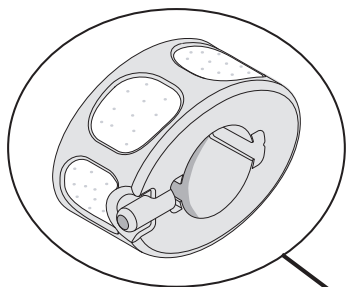
PROVOX[®] BasePlate Adaptor Life

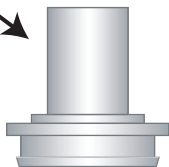
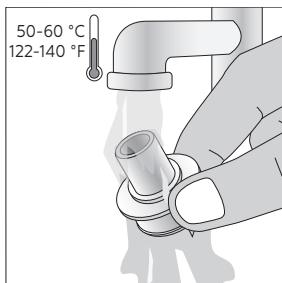
ISO 15 mm


TrachPhone®


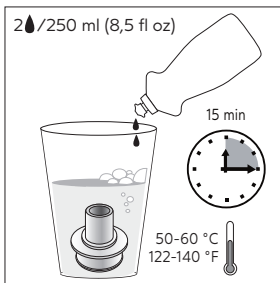
1.

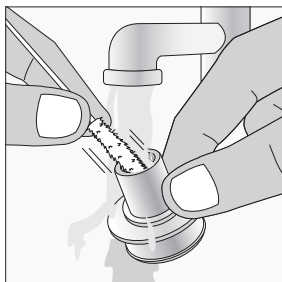
2.

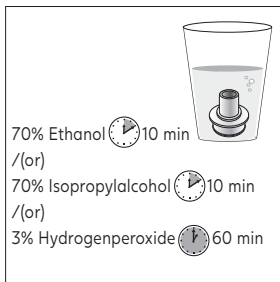3.

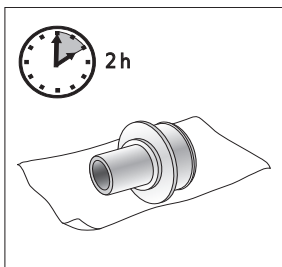4.

5.

4

Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Compatible Products; Kompatible Produkte; Compatibele producten; Produits compatibles; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Kompatible produkter; Yhteensopivat tuotteet; Samhæfar vörur; Úhilduvad tooted; Saderīgie izstrādājumi; Suderinami gaminiai; Kompatibilní produkty; Kompatibilis termékek; Združljivi izdelki; Kompatibilné produkty; Zgodne produkty; Prognose compatibile; Kompatibilni proizvodi; Kompatibilni proizvodi; Συμβατά προϊόντα; Uyumlu Ürünler; Համատեղելի սարքեր; Совместимые изделия; Produk yang Sesuai; 互換性のある製品; 호환 가능한 제품; 相容產品; 兼容产品; מוצרים תואמים; المنتجات المتوافقة

Provox Life LaryTube
Provox Life LaryButton
Provox Life Standard Adhesive
Provox Life Sensitive Adhesive
Provox Life Night Adhesive
Provox Life Stability Adhesive
TrachPhone
Freeevent DualCare
Freeevent XtraCare
ISO 15 HMEs

Contents

EN - ENGLISH	7
DE - DEUTSCH	9
NL - NEDERLANDS	11
FR - FRANÇAIS	13
IT - ITALIANO	15
ES - ESPAÑOL	17
PT - PORTUGUÊS	19
SV - SVENSKA	21
DA - DANSK	23
NO - NORSK	25
FI - SUOMI	27
IS - ÍSLENSKA	29
ET - EESTI	31
LV - LATVIESU	33
LT - LIETUVIŲ KALBA	35
CS - ČESKY	37
HU - MAGYAR	39
SL - SLOVENŠČINA	41
SK - SLOVENČINA	43
PL - POLSKI	45
RO - ROMÂNĂ	47
HR - HRVATSKI	49
SR - SRPSKI	51
EL - ΕΛΛΗΝΙΚΑ	53
TR - TÜRKÇE	55
HY - ՀԱՅԹԻՒՆ	57
RU - РУССКИЙ	59
MS - BAHASA MELAYU	61
JA - 日本語	63
KO - 한국어	65
ZHTW - 繁體中文	67
ZHCN - 简体中文	69
HE - עברית	72
AR - العربية	74

Intended use

Provox Life™ BasePlate Adaptor is an accessory that allows attaching medical devices, e.g. a HME, with an ISO 15 mm standard connector to a Provox Life™ attachment.

Contraindications

Shall not be used for mechanical ventilation.

Description of the device

Provox Life™ BasePlate Adaptor is an accessory that allows attaching and detaching medical devices, e.g. an HME, with an ISO 15 mm standard connector to a Provox Life™ attachment.

WARNINGS

- Unconscious patients must be constantly monitored. Attached devices may inadvertently slide off the adaptor, or the baseplate may detach from the tracheostoma.
- Make sure the adaptor is free from oil or any other lubricant before and during use. Lubrication may lead to detachment of devices from the adaptor, or of the adaptor from the baseplate.

PRECAUTIONS

- Read and follow the instructions of the medical device to be used with the adaptor.
- Never use force while attaching or detaching devices or the adaptor itself.
- Do NOT sterilize! The device is for single patient use only.
- Do not use if the device shows any signs of cracking or structural damage.

Cleaning and maintenance

Clean and disinfect the device before each use. Use drinking water to clean and rinse the device.

- Rinse the device in water (Fig. 1).
- Place the device in water-based mild soap solution for 15 minutes (Fig. 2).
- Clean inside of the device with Provox TubeBrush or Provox Swab. Rinse the device with water (Fig. 3).
- Disinfect the device at least once a day with one of the following methods: Ethanol 70% for 10 minutes or Isopropylalcohol 70% for 10 minutes or Hydrogenperoxide 3% for 60 minutes (Fig. 4).
- Let the device dry at least for 2 hours or until it looks dry (Fig. 5).

CAUTION: Do not use the device until it is completely dry. Inhalation of disinfectant fumes can cause severe coughing and irritation of the airways. The device shall be cleaned and disinfected before each use to avoid risk of contamination. The device is for single patient use only, reuse between patients may cause cross contamination.

During hospitalization it is important to clean and disinfect the device immediately after use and again just before use due to an increased risk of infection and device contamination. Use sterile water for rinsing, rather than drinking water.

Device lifetime

Provox Life™ BasePlate Adaptor may be used for a maximum period of 3 months. If the device shows any signs of damage it shall be replaced earlier.

Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.
Atos Medical AB