






Combining a speaking valve with the strengths of an HME - with a simple twist

Twist between speaking mode to speak and HME mode to take care of your lungs

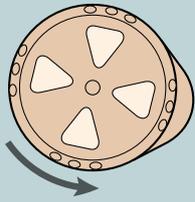



Speaking mode

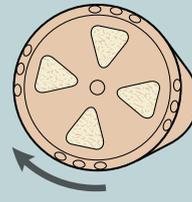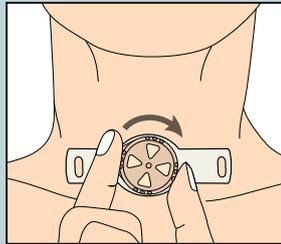




HME mode

To improve or maintain your lung health, you should use the HME mode as much as possible, which means whenever you are not speaking.









How to assemble the HME and Freevent DualCare Speaking Valve / HME DigiTop

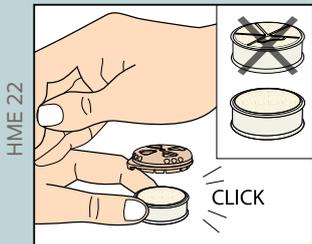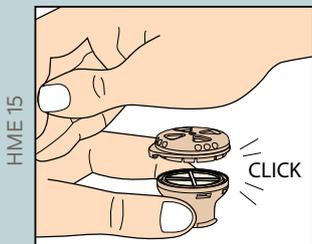






For maximum respiratory care, an HME should be used day and night. During daytime with the speaking valve and while you sleep with the HME DigiTop, that also allows finger occlusion for speaking.

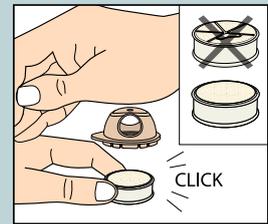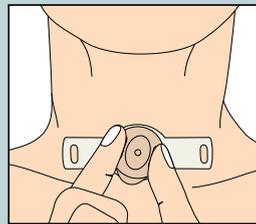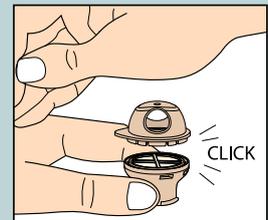


How to remove the Freevent HME

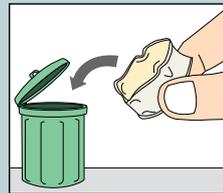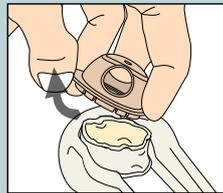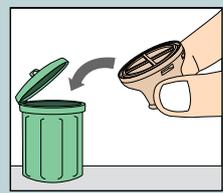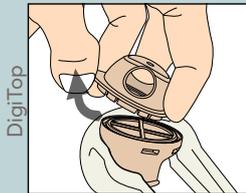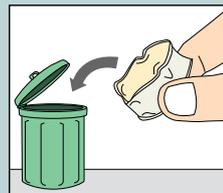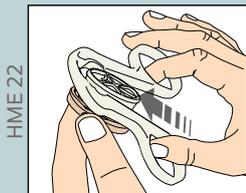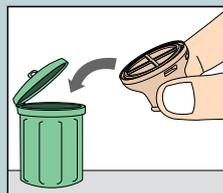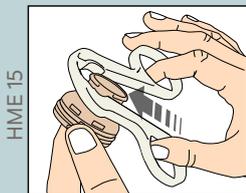


## How to attach the Freevent Connection Strap

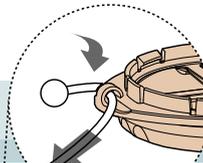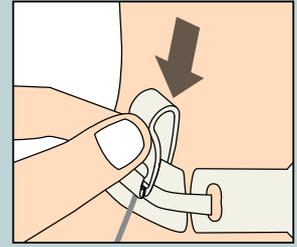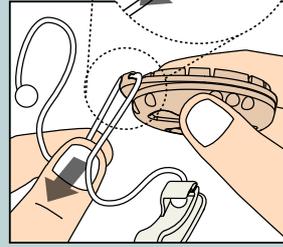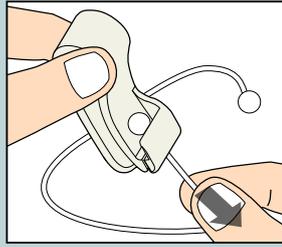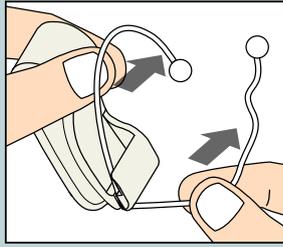


## How to clean the Freevent DualCare Speaking Valve

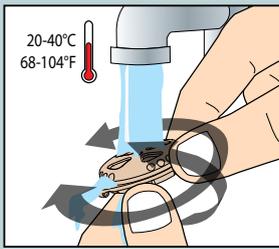


2 min

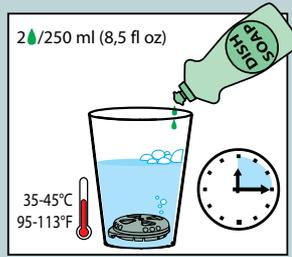


15 min

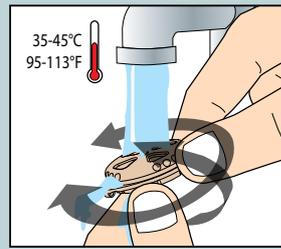

15 sec


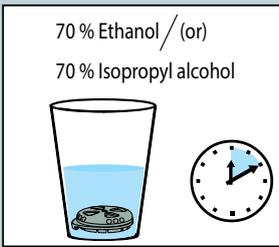


10 min


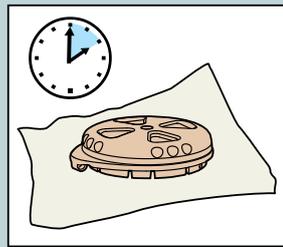


2 h



## DO NOT:

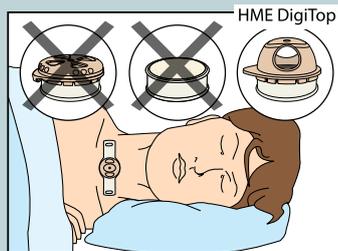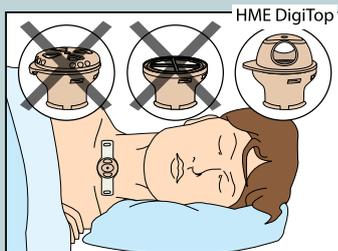

DO NOT use the Freevent Speaking Valve during sleep since the airway could be blocked unintentionally. Use the HME DigiTop instead.

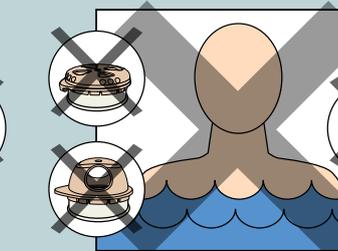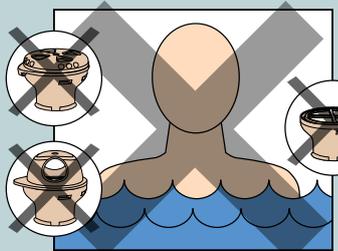

DO NOT bathe or swim with the device. This could cause water to enter the airways.

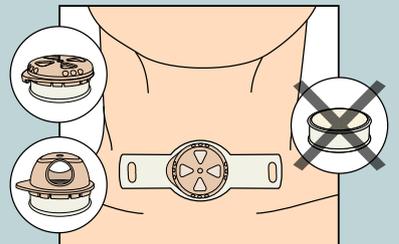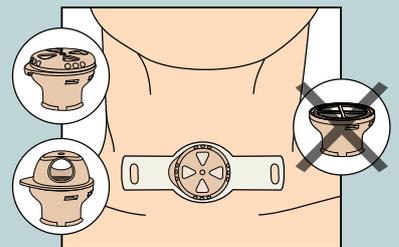

Do not use the HME alone, because it can be difficult to remove.

Please note: Freevent DualCare Speaking Valve can only be used with tracheostomy tubes if you have free passage for air through the upper airways.

Always read the Instructions for use before starting to use any of the products. For more information about the Freevent assortment and Instructions for use, please visit [www.atosmedical.com](http://www.atosmedical.com).